

MAESTRO UPDATE AVAILABLE

Printing Issue Resolution

ENGLISH

Dear Customer,

We would like to inform you that **Maestro 1.28.0 is now available** and includes a fix for the **printing and PDF export issues** experienced after certain *Windows updates*.

If you are using a Maestro version earlier than 1.28.0, please update the software to version 1.28.0.

Important for Celesta users: We recommend waiting before updating Maestro until the new Celesta software becomes available, which is expected in week 39. Updating Maestro before then will cause Celesta not to be recognized by the new version.

[DOWNLOAD THE LATEST VERSION OF MAESTRO](#)

<https://inventis.owncloud.online/s/6PsZzrdM9oXOliA>

After downloading the installer, proceed with the Maestro update by following the on-screen instructions.

For any questions or assistance, our Customer Care team remains at your full disposal.

Kind regards,
Inventis Customer Care